



Non Contact Tono / Pachymeter

TONOPACHY™ NT-530P



THE ART OF EYE CARE

Non Contact Tono / Pachymeter
TONOPACHY™ NT-530P

Enhanced Combination System

Succeeding the TONOREF™II,
Auto Ref / Kerato / Tonometer,
NIDEK realizes the enhanced combination
system again in the TONOPACHY™ NT-530P,
the unique model combining Non Contact
Tonometer and Pachymeter in one unit.



■ Auto and Non Contact Pachymeter

Taking an advantage of the principle of Scheimpflug camera, the TONOPACHY™ NT-530P provides non contact measurement of patient's central corneal thickness. With its smart 3-D auto tracking and auto shot, the TONOPACHY™ NT-530P allows safe, easy, and accurate measurement on the Pachymeter.





■ Automatic Calculation of Compensated IOP

The TONOPACHY™ NT-530P provides the automatic calculation function of the compensated IOP based on the central corneal thickness. This value can be used for the reference with consideration of patient's corneal thickness.



■ Softer and Quieter Air Puff


The NIDEK innovative technology enhances APC (Automatic Puff Control) to the next stage. It reduces the noise and air intensity. With the enhanced APC, the TONOPACHY™ NT-530P achieves a softer and quieter air puff than ever, which allows the patient's more comfort.

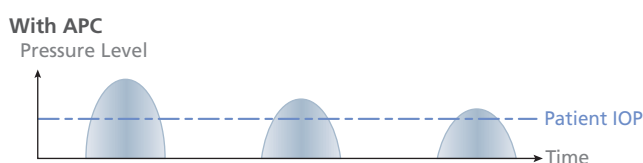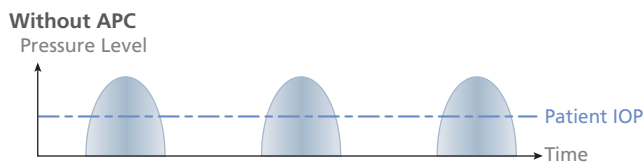

■ 3-D Auto Tracking, Auto Shot, and Auto Complete

Based on a long experience in developing examination equipment, NIDEK incorporates the unique auto complete in the Non Contact Tonometer. In the AI (Artificial Intelligence) mode, the TONOPACHY™ NT-530P automatically detects the most appropriate value and completes the measurement.

The combination of auto complete with the world renowned 3-D auto tracking and auto shot provides the operator with the most ease, comfort, and accuracy on all measurements.

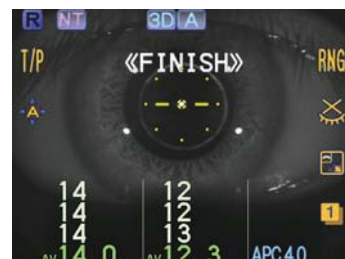

TONOPACHY™ NT-530P Specifications

Non contact tonometer	
Measurement range	1 to 60 mmHg
Measurement range setting	APC40, APC60 (APC=Automatic Puff Control), 40, 60
Working distance	11 mm
Eye fixation	Inner fixation light
Non contact pachymeter	
Measurement range	150 to 1300 μm
Light source	Blue LED
IOP compensation by corneal thickness	Automatic calculation
Auto tracking / Auto shot	X-Y-Z direction Auto shot
Auto complete	Available
Display	Tilttable 5.7-inch color LCD
Printer	Thermal line printer with easy loading and auto cutter
Interface	RS-232C, LAN, USB
Power supply	AC 100 to 240 V ±10%, 50 / 60 Hz
Power consumption	100 VA
Dimensions / Mass	260 (W) x 481 (D) x 474 (H) mm / 20 kg 10.2 (W) x 18.9 (D) x 18.7 (H)" / 44 lbs.

Product / Model name: Noncontact Tono / Pachymeter NT-530P

Specifications may vary depending on circumstances in each country.

Specifications and design are subject to change without notice.



HEAD OFFICE
(International Div.)
34-14 Maehama, Hiroishi
Gamagori, Aichi 443-0038,
Japan
TEL: +81-533-67-8895
URL: <http://www.nidek.co.jp>
[Manufacturer]

TOKYO OFFICE
(International Div.)
3F Sumitomo Fudosan Hongo
Bldg., 3-22-5 Hongo, Bunkyo-ku,
Tokyo 113-0033, Japan
TEL: +81-3-5844-2641
URL: <http://www.nidek.com>

NIDEK INC.
47651 Westinghouse Drive
Fremont, CA 94539, U.S.A.
TEL: +1-510-226-5700
+1-800-223-9044
(US only)
URL: <http://usa.nidek.com>

NIDEK S.A.
Europarc
13, rue Auguste Perret
94042 Créteil, France
TEL: +33-1-49 80 97 97
URL: <http://www.nidek.fr>

NIDEK TECHNOLOGIES Srl
Via dell'Artigianato,
6 / A 35020 Albignasego
(Padova), Italy
TEL: +39 049 8629200 / 8626399
URL: <http://www.nidektechnologies.it>

NIDEK (SHANGHAI) CO., LTD.
Rm 915, China Venturetech
Plaza, No.819 Nanjing West Rd,
Jing An District, Shanghai
China 200041
TEL: +86 021-5212-7942
URL: <http://www.nidek-china.cn>

NIDEK SINGAPORE PTE. LTD.
51 Changi Business Park
Central 2 #06-14
The Signature Singapore
486066
TEL: +65 6588 0389

